Bionova® IC10/20FRLCD Auto-reader

Symbols | Símbolos | Símbolos | Simboli

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EN Bionova® IC10/20FRLCD Auto-reader

Product description

The Bionova® IC10/20FRLCD Auto-reader was designed for the incubation and automatic readout of Terragene® Bionova® Rapid, Super Rapid and Ultra Rapid Self-contained Biological Indicators (SCBIs) for sterilization and environmental disinfection process control applications, and for the incubation and automatic readout of Terragene® Chemdry® Hygiene Monitoring System Indicators (Protein pens) for cleaning control applications. The Bionova® IC10/20FRLCD Auto-reader incubates at two different temperatures, 37 °C or 60 °C. The microorganisms within SCBIs vary depending on the sterilization process they were designed for; therefore, incubation temperatures and times differ accordingly. The Bionova® IC10/20FRLCD Auto-reader allows independent incubation time selection for each incubation position. SCBIs designed for ethylene oxide (EO) sterilization are incubated at 37 °C, while those used for monitoring other sterilization processes are incubated at 60 °C. **Note:** Simultaneous incubation of SCBIs for monitoring EO processes along with SCBIs for monitoring other sterilization processes is not possible.

The Bionova® IC10/20FRLCD Auto-reader detects positive and negative SCBIs easily and quickly, using advanced fluorescence techniques. A positive result can also be shown by a change in the culture medium color when extended incubators are carried out. Read the Instructions For Use of the SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital. The Bionova® IC10/20FRLCD Auto-reader generates a printed ticket each time an incubation program has been completed enabling easy result management, for documentation compliance and safe-keeping.

Indications for Use

United States of America: Terragene® Bionova® Reader Incubators IC10/20FRLCD incubate at 60 °C and 37 °C and read the Terragene® Bionova® SCBI for fluorescent results at the times prescribed in the User Manuals. **Outside the United States of America:** Bionova® Reader Incubators IC10/20FRLCD incubate at 60 °C and 37 °C, and read the Bionova® SCBIs and the Chemdry® Hygiene Monitoring System indicators at the times prescribed in the User Manuals.

Safety Information

- To avoid risks and/or damaging the device:**
 - Use indoors only.
 - Do not place the auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
 - Do not place the auto-reader near devices that emit strong electromagnetic fields.
 - Do not use the auto-reader on sloping surfaces or on surfaces that may be knocked, that may vibrate, be exposed to high temperatures or high relative humidity.
 - Disconnect the power cord before cleaning.
 - Do not use abrasive or corrosive cleaners or disinfectants.
 - Do not immerse into any liquid. Do not pour liquid into it.
 - Make sure the auto-reader is connected to an appropriate electrical mains outlet socket.
 - Use the included power supply (AC power adapter), power supplys AC plug, power supply cords and Ethernet cable only. Check that all the items included are in good condition every day. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies differs from the ones included may cause fires, electrical shocks, or even physical injuries.
 - Do not plug any devices into the auto-reader's Ethernet port other than devices compliant with IEC 60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its Ethernet ports. Ask a qualified technician to check that the device is compatible. Attaching any other device to the Ethernet port may damage the auto-reader and may not be safe for the user.
 - Do not attempt to repair the auto-reader yourself. This could lead to major and irreversible damages to the device. If the auto-reader malfunctions, contact your local distributor for further assistance.

- To reduce the risk of using incompletely sterilized loads:**
 - Please read, understand, and follow the Instructions For Use of each SCBI before incubating it.
 - Do not remove the SCBI before the auto-reader shows the final readout result.
 - Check that the spore carrier has been wetted completely by the culture medium.

- To avoid the risk of injury by glass fragments produced when crushing the glass ampoule inside the SCBI tube:**
 - Cool the SCBI during the incubated time before crushing the ampoule.
 - Do not handle the SCBI excessively since this might cause the glass ampoule to explode.
 - Wear safety gloves and goggles when removing the SCBI from the sterilizer, pressing the SCBIs cap, and crushing the SCBIs ampoule.
 - Do not use your finger to crush the SCBI. Use the ampoule crusher instead.

- To avoid a potentially hazardous situation:**
 - Avoid contact with the hot metal block inside each incubation position.
 - Do not insert your fingers, or any other element, into the incubation positions.
 - Place only compatible SCBIs inside the incubation positions.

- To avoid SCBIs from absorbing fluorescent particles:**
 - Avoid direct contact between the SCBIs and chemical indicators or tapes before the SCBI incubation.
 - Avoid excessive SCBI handling that may lead to fingerprints or glove talc imprinted over the SCBI.

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Note: Only Terragene S.A. authorized personnel may access or service the internal components of the auto-reader. Parts or components inside the auto-reader should not be handled by the user.

Operating conditions

Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts

Input parameters	Operating conditions	Units
Frequency	50-60	Hertz
Current	1	Ampere
Output parameters	Values	Units
Voltage	12	DC Volts
Current	3	Ampere

Terragene S.A. recommends the using Interruptible Power Supply (UPS) instead of voltage stabilizers, since they fulfill two functions: they stabilize and maintain the energy during a power outage.

Environmental operating conditions	Operating conditions	Units
Altitude	3500 (maximum)	Meters
Operation temperature	10-30	Celsius
Relative humidity	30-80	%
Installation/Over-voltage	Category II	-
Degree of contamination	2	-
Storage temperature	5-40	Celsius
Voltage	12	DC Volts
Device safety compliance The Bionova® IC10/20FRLCD Auto-reader complies with the following standards and directives:		
Electrical Safety	IEC 61010-1 IEC 61010-2-010 Low Voltage Directive 2014/35/EU	
Electromagnetic Compatibility (EMC)	EN 61326-1 EMC Directive 2014/30/EU	
European Commission	RoHS Directive 2011/65/EU WEEE Directive 2012/19/EU	

The Bionova® IC10/20FRLCD Auto-reader does not represent photobiological risk and does not generate dangerous optical radiation used within its normal operating conditions, as per the requirements of IEC 62471 Standard. **Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.** **Note:** The auto-reader has been evaluated for conformity for use in business environments. There may be concerns about radio interference if used in home environments. This applies exclusively to the Korean market due to KC certification requirements.

Cleaning and maintenance
Cleaning and decontaminating of external surfaces
Disconnect the power supply cable and the Ethernet cable from the auto-reader. If the device is hot, wait until it has cooled down before handling it. Clean the external surfaces of the device using a microfibr cloth moistened with a solution of mild dish washing detergent and water. Wring out the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the auto-reader. Then, moisten a clean microfibr cloth with just water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the auto-reader to air dry in air for at least 1 hour before connecting the power supply cable or the Ethernet cable again. This cleaning procedure can be followed whenever considered appropriate. This cleaning procedure must be followed each time a spill occurs on any of the external surfaces of the auto-reader. If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor. **Note:** Do not clean the internal parts of the auto-reader. **Note:** Do not pour liquid into the device or immerse the it in any liquid. Do not allow any liquid to enter device while it is being cleaned.

Maintenance
The auto-reader does not require routine maintenance.

Warranty
The products are guaranteed to be free from material and workmanship defects when properly installed, maintained and used for their intended purpose as indicated in the applicable product label and/or the User Manual. The warranty only applies to the original purchaser.

Term
The warranty period for the Bionova® IC10/20FRLCD Auto-reader is 1 (one) year from the date of installation and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability
Terragene S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
For information regarding the functioning or condition of the product, contact Terragene S.A. directly to info@terragene.com or through our website www.terragene.com. In USA, please call +1 844 837 7243.

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EN Instructions for use

Note: Remember to set the auto-reader local date and time before starting incubations.

Setting up your auto-reader

Place the auto-reader on a firm surface, free from vibrations, away from direct sunlight, hot or cold air flows, chemical and corrosive or flammable substances. Do not place the auto-reader in a way that would make it difficult to remove the power supply's AC plug from mains. Leave a space of at least 10 cm from the auto-reader to the closest wall. Do not move the auto-reader periodically or during its use. Connect the auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the device. If liquid is spilled on the auto-reader, disconnect and clean it. Before switching it on, check that all incubation positions are empty.

Using your auto-reader

- 1 Switch on the auto-reader by connecting the power supply's AC plug to the mains and then connect the plug at the other end of power supply to the rear of the auto-reader (See image 1).
- 2 Set the auto-reader local date and time. For more information, please refer to the User Manual.
- 3 Press and hold the temperature selection button for 3 seconds (See image 2). The temperature modification will automatically be accepted. Wait until the temperature & stability indicator light stops blinking.
- 4 A) SCBI Incubation procedure
4.A.1 Select the appropriate incubation time for the incubation position you are planning to use. Press the SCBI position configuration button on the main screen. Then, press the program configuration button and choose the SCBI from the list of compatible SCBIs. Once selected, the auto-reader will automatically set the incubation time for the chosen SCBI (See image 3.A).
4.A.2 Place an SCBI in an incubation position. The reading process will start automatically after placing the SCBI (See image 3.B).
- 4.B) PRO1 Indicator Incubation procedure
4.B.1 Access the PRO1 configuration screen (See image 4.A).
4.B.2 Place the PRO1 indicator fully inside the PRO1 incubation position (See image 4.B). Start the incubation and readout process by pressing the button.
- 5 Wait until a readout result has been carried out. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. For PRO1 indicators, the auto-reader will perform the equivalent µg of protein quantity using a BSA (Bovine Serum Albumin) calibration curve. Do not remove the SCBI / PRO1 indicator once the reading process has begun. If this occurs, the results may be invalidated.

Note: To select a protein quantification threshold mode and a threshold value, please refer to the User Manual.

Disposal: Dispose of the SCBIs in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your IC10/20FRLCD Auto-reader to a personal computer (PC) by Ethernet, connect the auto-reader to an Ethernet switch or router using a suitable Ethernet cable. Then, connect your PC to the same Ethernet switch or router using a suitable Ethernet cable. Connect the auto-reader to a PC to use the Bionova® Cloud Web Application. Please refer to the User Manual for more information.

Bionova® IC10/20FRLCD Auto-reader incubation programs

The following incubation programs are available for your auto-reader.

Temperature	Time	Incubation program
60 °C	20 minutes	20 minutes at 60 °C
	30 minutes	30 minutes at 60 °C
	1 hour	1 hour at 60 °C
	2 hours	2 hours at 60 °C
	3 hours	3 hours at 60 °C
	PRO (7 minutes)	PRO at 60 °C
37 °C	4 hours	4 hours at 37 °C

Note: Please read and follow the SCBI/ PRO1 indicator Instructions For Use before using it.

Troubleshooting

Refer to the User Manual for detailed troubleshooting measures.

Please visit www.terrargene.com/IC1020FRLCD or scan the QR code to access the auto-reader User Manual.

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