Bionova® MiniBio Auto-reader

Symbols | Símbolos | Símbolos | Simboli

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EN Bionova® MiniBio Auto-reader

Product description
Bionova® MiniBio Auto-reader has been designed for the incubation and automatic reading of Terragene® Rapid, Super Rapid and Ultra Rapid Self-contained Biological Indicators (SCBIs) appropriate for sterilization and airborne and surface disinfection process control applications.
Bionova® MiniBio Auto-reader allows two different incubation temperatures, 37 °C or 60 °C. Microorganisms contained within the SCBIs vary depending on the sterilization process for which they have been designed. Therefore, incubation temperatures and incubation times differ depending on the SCBI being used. Bionova® MiniBio Auto-reader allows independent time settings selection for each one of the three incubation positions. SCBIs for ethylene oxide (EO), should be incubated at 37 °C, while those used for the monitoring of the rest of the sterilization processes should be incubated at 60 °C.
Non simultaneous incubation of SCBIs for monitoring EO processes along with SCBIs for monitoring other sterilization processes is not possible.
Bionova® MiniBio Auto-reader allows easy and rapid detection of positive and negative SCBIs using advanced fluorescence techniques. A positive result can also be evidenced by culture medium color change when performing extended incubations. Read the instructions for use of each SCBI for more information. The option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital.
Bionova® MiniBio Auto-reader also provides a printed ticket each time an incubation is completed for recording the results. This allows for easy result management, for documentation compliance and easy-keeping.

Indications for Use
Terragene® Bionova® Reader Incubators MiniBio incubate at 60 °C and 37 °C and read the Terragene® Bionova® SCBI for fluorescent results at the times prescribed in the User Manuals.

Safety Information
 **To avoid risks and/or damaging the device:**
-Do not use only.
-Do not place the Auto-reader in a room exposed to direct sunlight or to high luminous intensity lamps.
-Do not place the Auto-reader near devices that emit strong electromagnetic fields.
-Do not use the Auto-reader on sloping surfaces or on surfaces that may be knocked, that may vibrate, be exposed to high temperatures or high relative humidity.
-Disconnect the power cord before cleaning.
-Do not use abrasive or corrosive cleaners or disinfectants.
-Do not immerse into any liquid. Do not pour liquid into it.
-Make sure the Auto-reader is connected to an appropriate electrical mains outlet socket.
-Use the included power supply (AC power adapter), power supply AC plug, power supply cords, and USB cable only. Check that all the items included are in good condition every day. If any of them is damaged, discontinue their use. The use of cables, adapters, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.
-Do not attempt to repair the Auto-reader by yourself. That could lead to major and irreversible damages to the Auto-reader. In case of device malfunction, contact your local distributor for further assistance.
-Do not plug any devices into the Auto-reader's USB port other than a personal computer (PC). The PC must be compliant with IEC 60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on the USB ports. Ask a qualified technician to check that the device is compatible. Attaching any other device to the USB port may damage the Auto-reader and may not be safe for the user.
-Do not attempt to repair the Auto-reader by yourself. This could lead to major and irreversible damages to the device. If the Auto-reader malfunctions, contact your local distributor for further assistance.

 **To reduce the risk of using incompletely sterilized loads:**
-Please read, understand, and follow the Instructions for Use of the SCBI before incubating it.
-Do not remove the SCBI before the Auto-reader shows the final readout result. Check a result ticket is printed.
-Check that the spore carrier has been wetted completely by the culture medium.

 **To avoid the risk of injury by glass fragments produced when crushing the glass ampoule inside the SCBI tube:**
-Cool the SCBI during the indicated time before crushing the ampoule.

Do not handle the SCBI excessively since this might cause the glass ampoule to explode.
-Wear safety gloves and goggles when removing the SCBI from the sterilizer, pressing the SCBI's cap, and crushing the SCBI's ampoule.
-Do not use your finger to crush the SCBI. Use the ampoule crusher instead.

 **To avoid a potentially hazardous situation:**
-Avoid contact with the hot metal block inside each incubation position.
-Do not insert your fingers, or any other element, into the incubation positions.
-Place only compatible SCBIs inside the incubation positions.

 **To avoid SCBIs from absorbing fluorescent particles:**
-Avoid direct contact between the SCBIs and chemical indicators or tapes before the SCBI incubation.
-Avoid excessive SCBI handling that may lead to fingerprints or glove talc imprinted over the indicator.

Note: do not use this product in a manner not specified by Terragene S.A., otherwise the protection provided by the product might be affected.
Note: only Terragene S.A. authorized personnel may access or service the internal components of the Auto-reader. Parts or components inside the Auto-reader should not be handled by the user.

Operating conditions Power supply specifications		
Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts
Frequency	50-60	Hertz
Current*	0.5	Amperes
Output parameters	Values	Units
Voltage	12	DC Volts
Current**	2	Amperes
* 1 Ampere for UL listed devices. ** 3 Amperes for UL listed devices. Terragene S.A. recommends using Uninterruptible Power Supply (UPS) instead of voltage stabilizers, since they fulfill two functions: they stabilize and maintain the energy during a power outage.		
Environmental operating conditions		
Environmental conditions	Operating conditions	Units
Altitude	3500 (max.)	Meters
Operation temperature	10-30	Celsius
Relative Humidity	30-80	%
Installation/Over-voltage	Category II	-
Pollution degree	2	-
Storage temperature	5-40	Celsius
Voltage	12	DC Volts
Device safety compliance		
Bionova® MiniBio Auto-reader complies with the following standards and directives:		
Electrical Safety	IEC 61010-1 IEC 61010-2-010 Low Voltage Directive 2014/35/EU	
Electromagnetic Compatibility (EMC)	EN 61326-1 EMC Directive 2014/30/EU	
European Commission	RoHS Directive 2011/65/EU WEEE Directive 2012/19/EU	

Bionova® MiniBio Auto-reader does not represent photobiological risk and does not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471 standard. Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.
Note: the Auto-reader has been evaluated for conformity for use in business environments. There may be concerns about radio interference if used in home environments. This applies exclusively to the Korean market due to KC certification requirements.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the USB cable from the Auto-reader. If the Auto-reader is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device using a microfibr cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the Auto-reader. Afterwards, moisten a clean microfibr cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Auto-reader to cool for at least 10 minutes before connecting the power supply cable or the USB cable again. This cleaning procedure can be followed whenever considered appropriate.
This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the Auto-reader.
If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your distributor for further assistance.

Note: do not clean the internal parts of the Auto-reader.
Note: do not pour or immerse the Auto-reader into any liquid. Do not allow any liquid to run inside the device during its cleaning.

Maintenance
The Auto-reader does not require routine maintenance.

Warranty
The products are guaranteed to be free from material and workmanship defects when properly installed, maintained and used for their intended purpose as indicated in the applicable product label and/or the User Manual. The warranty only applies to the original purchaser.

Term
The warranty period for the Bionova® MiniBio Auto-reader is 1 (one) year from the date of installation and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability
Terragene S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
Terragene S.A.
Ruta Nacional N° 9, Km 280 - CP 2130.
Parque Industrial Microipi-Alvear-Santa Fe-Argentina.

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EN Instructions for use

Note: remember to set-up the Auto-reader local date and time before starting incubations.

Setting Up your Auto-reader

Place the Auto-reader on a firm surface, free from vibrations, away from direct sunlight, hot or cold air flows, chemical and corrosive or flammable substances. Do not place the Auto-reader in a way that would make it difficult to remove the power supply's AC plug from mains. Leave a space of at least 10 cm from the Auto-reader to the closest wall. Do not move the Auto-reader periodically or while it is being used. Connect the Auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto-reader. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all incubation positions are empty.

Using your Auto-reader

- Switch on your Auto-Reader by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto-Reader. (See image 1)
- Press and hold down the temperature selection button for 3 seconds to enter the temperature selection mode. Press again to select the desired temperature. After 4 seconds the temperature modification will be accepted automatically. (See image 2)
- Wait until the temperature & stability indicator light stops blinking. Press the incubation time selection button to select the appropriate incubation time for the SCBI you are planning to use, according to the SCBI's instructions for use.
- Place an SCBI in an SCBI incubation position. The incubation process will start automatically. (See image 3)
- Wait until the readout result has been carried out. If a negative result is detected in an incubation position, the negative indicator light will turn on (green). If a positive result is detected in an incubation position, the positive indicator light will turn on (red) and an audible alarm will sound. Press the alarm cancellation button to cancel the alarm. Do not remove the SCBI until the result has been produced. Disposal: dispose of the SCBIs in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your Auto-reader to a personal computer (PC) by USB, connect an end of the USB cable provided to the rear part of the Auto-reader and the other end to an USB port on your PC. Connect the Auto-reader to a PC to use the Bionova® Cloud Environment. Please refer to the User Manual for more information.

Product Compatibility and Incubation Programs

The following incubation programs are available for your Auto-reader.

Temperature selection	Time selection	Incubation program
60 °C	20 minutes 30 minutes 1 hour 2 hours 3 hours	20 minutes at 60 °C 30 minutes at 60 °C 1 hour at 60 °C 2 hours at 60 °C 3 hours at 60 °C
37 °C	4 hours	4 hours at 37 °C

To select the appropriate incubation program for the SCBI you are planning to use, follow each SCBI's instructions for use.

Note: select the appropriate incubation program before the incubation of any SCBI.

Note: all SCBIs are single use. Do not use or incubate the same SCBI more than once.

Troubleshooting

Refer to the User Manual for a detailed troubleshooting measures. Please visit www.terrakene.com/MINIBIO or scan the QR code to access the Auto-reader User Manual.

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