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## WiFi connectivity accessory for Bionova® incubators with Ethernet port.

### Product description

WILINK allows an Incubator with Ethernet port to connect to WiFi networks.

### Characteristics

Dimensions: 15 mm high, 93 mm wide, 47 mm deep  
 Voltage: 12 V DC  
 Power: 5 W  
 Connections: Ethernet / WiFi  
 Storage conditions: T 10-30 °C, HR 30-80 %

### Ethernet Characteristics:

- 10/100 Base Ethernet

### WiFi Characteristics:

- Transmission protocol: 802.11 b/g/n  
 - Cryptographic Protocols: WPA-PSK/WPA2-PSK, 64/128/WEP  
 - Frequency: 2.4 GHz  
 - Transmission Speed: 300 Mbps  
 - Point to point transmission distance: 80 m – 100 m

### WARNING

Description of identified and/or predictable risks which could represent a risk for the product :

- Do not pour any liquid inside.
- Do not immerse into any liquid.
- Only for indoor use.
- Disconnect the power cord before cleaning.
- Do not use abrasive or corrosive cleaners or disinfectants.
- Use the cables included in the product box.
- Make sure that the incubator is connected to a properly rated power cord.

### Directions of use

In order to use this device, a computer with an Ethernet port is needed. To configure it, use the WILINK Wizard tool included in the Bionova® Traceability Software (version 3.0 or higher). It can also be downloaded from <http://www.terragene.com.ar/productos/infection-control/incubators/softw-are-downloads/?lang=en>. To configure the device, execute the WILINK Wizard tool and follow these steps:

1. Connect the WILINK to your computer's Ethernet port.
2. Energize the WILINK and wait 60 seconds.
3. Select from the list the network you want to connect to and enter the password if needed.

-If the chosen network is not on the list, update the list with the Reset button.

-If after several tries no networks are shown, check the connections and start over the configuring process.

**NOTE:** To configure the device when previously configured, default parameters must be restored. To do this, press Reset button for 5 seconds. When released, the blue light will blink, then the device will automatically reset to the factory default parameters. This process will take about 80 seconds to complete.

4. If the procedure has been successful, connect the WILINK and the incubator using the cable included in the product. **NOTE:** You may connect several ethernet incubators with a single WILINK using an ethernet switch. For information on connection types, refer to the configuration wizard.

### Warranty

Terragene® S.A. guarantees both the quality of the product material components and the quality of its manufacturing process. Should any material or manufacturing faults be detected within the warranty period (6 months from the time of purchase), the only obligation of Terragene® S.A. will be product repairing or substitution.

### Limitation of liability

Terragene® S.A. shall not be liable for any loss or damage that may result from the unsuitable use of the equipment, negligence or user's full responsibility.