

SARS-CoV-2

Antigen Rapid Test Kit

(Colloidal Gold Immunochromatography)



- Non-invasive
- Simple to use
- Convenient, no devices required
- Rapid, get result in 15 minutes
- Stable, with high accuracy
- Inexpensive, cost-efficiency



Hospital



Test site



Airport



Station



Hotel




Corporation




Mass Screening




Instruction

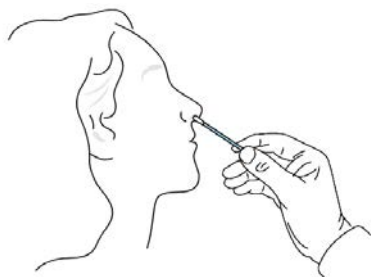


Step 1: Use swab to collect sample.

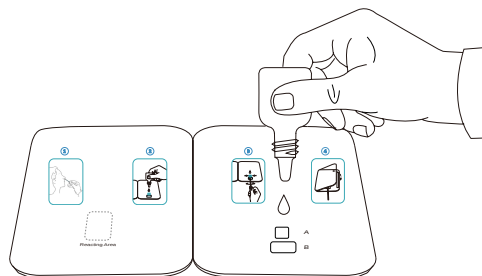

Step 2: Add 5 drops of buffer to well A.

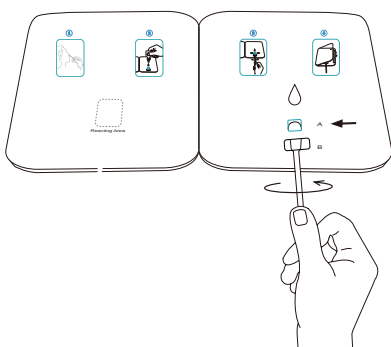

Step 3: Insert the sample through well B to well A, rotate the shaft clockwise first and counterclockwise, twice each.

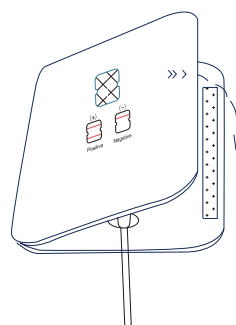


Step 4: Peel off the adhesive, fold the left side over, and wait for 15 minutes.



Result Interpretation

Positive

(+)



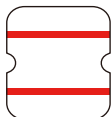


Negative

(-)




Invalid

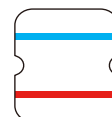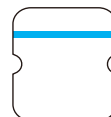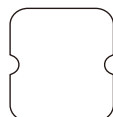

LEPU MEDICAL TECHNOLOGY (BEIJING) CO., LTD

No.37 Chaoqian Rd., Changping District, Beijing, China

SARS-CoV-2 Antigen Rapid Test Kit (Colloidal Gold Immunochromatography)

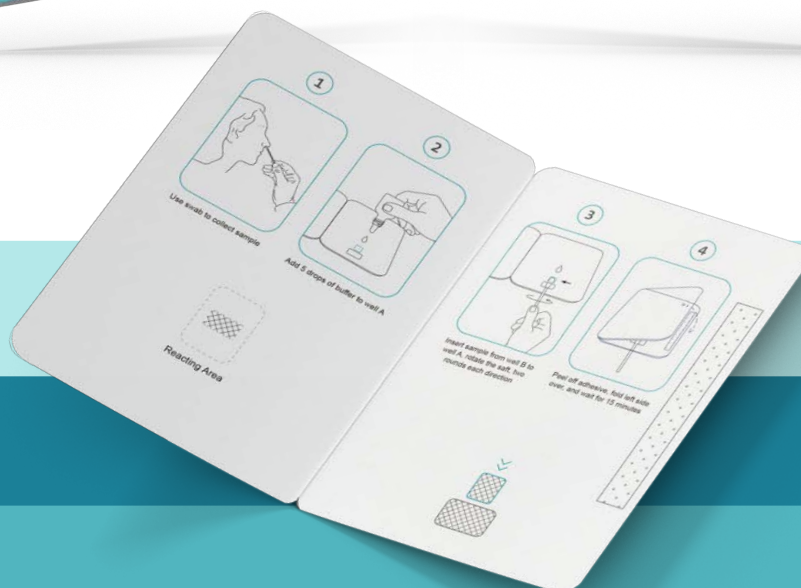

SARS-CoV-2 Antigen Rapid Test Kit

(Colloidal Gold Immunochromatography)



LEPU MEDICAL TECHNOLOGY (BEIJING) CO., LTD

No.37 Chaoqian Rd., Changping District, Beijing, China