In order to facilitate the registration of your Room Disinfection process monitoring, use this schematic documentation chart to highlight the position of the disinfection equipment and 3D Chemical indicators of the room you desire to disinfect. By using the Chemdye® ChemSurf Documentation Chart, you will be able to analyze the effectiveness of your room disinfection process.

References

- Use a circle to point out the location of your Room Disinfection equipment(s).
- Use a **X** (RED cross): to identify a POSITIVE RESULT=FAILED DISINFECTION of the Chemdye® ChemSurf CDS47 in that location.
- Use a **X** (GREEN cross): to identify a NEGATIVE RESULT=SUCCESSFUL DISINFECTION of the Chemdye® ChemSurf CDS47 in that location.
- Use a number next to each cross to differentiate between each 3D Chemical Indicator used and identify each one in the Results Table (behind).
- Use - - - - - (blue dotted line): to delineate the Risk Zone, according to your results.
- Draw other structures of your room, in order to locate yourself in this schematic chart.

NOTE: This chart is only intended to be used as a guideline for results registration. Use the squares of this chart according to the dimensions of your room.

RESULTS INTERPRETATION: The use of the Chemdye® ChemSurf CDS47 Documentation Chart will help the user identify the Risk Zones of the processed room and to proceed to take corrective actions to improve the Room Disinfection results. These corrective actions may include: change location of the Room Disinfection equipment, use more than one (1) Room Disinfection equipment, extend the exposure time, among others.

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