EN

Documentation Chart | CDSUV-1

For Registration of Disinfection Processes by Continuous UV-C Radiation (254 nm)

In order to ease the registration of your disinfection process, use this schematic Documentation Chart to indicate the positions of Chemdye® ChemSurf CDSUV-1 chemical indicators in the room. By using this Documentation Chart, you will be able to analyze the efficacy of your disinfection process.

References

- Use an **X** (RED cross): to identify a NEGATIVE RESULT, that is, a color change that corresponds to a received accumulated radiation dose lower than 50 mJ/cm².
- Use an **X** (GREEN cross): to identify a POSITIVE RESULT, that is, a color change that corresponds to a received accumulated radiation dose equal or higher than 50 mJ/cm².
- Use a number next to each cross to differentiate between each Chemdye® ChemSurf CDSUV-1 chemical indicator used and identify each one in the Results Table.
- Use - - - - - (blue dotted line): to delineate the risk zone, according to your results.
- Draw other structures of your room, in order to locate yourself in this schematic chart.

NOTE: this chart is only intended to be used as a guideline for results registration. Use the squares of this chart according to the dimensions of your room.

RESULTS INTERPRETATION: the use of this Documentation Chart will help the user identify the Risk Zones of the processed room and to proceed to take corrective actions to improve the results. These corrective actions may include, for example, extend the exposure time.

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