Infection Control Division | Biological Indicators

BT400 Spore strip biological indicator

For liquid peracetic acid high level disinfection processes

Usage

Monitoring the sporicidal activity of liquid peracetic acid in high level disinfection processes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/ EN ISO 13485:2016.

Authorization

ANMAT (Argentinian National Administration of Drugs, Food and Medical Devices) PM 1614-1.

Classification

Class 1, according to risk (ANMAT).


Characteristics

25.0 mm x 70.0 mm envelope of non-absorbent material. Turquoise code.

5.0 mm x 25.0 mm polyethylene fiber strip.

10^6 *Geobacillus stearothermophilus* ATCC®* 7953 spores per strip.

Silicone pull-tight seal: 80.0 mm long. Bead with hole: 5.6 mm diameter.

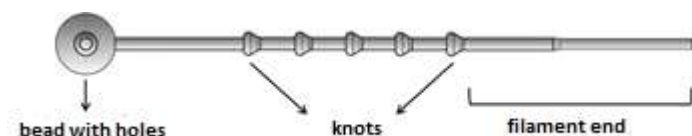Upon completion of the disinfection process, the spore strip should be tested in a suitable culture medium at 60 °C. We suggest the use of Bionova® MC1020-2 culture media to final reading in 24 hours. For conventional culture media like TSB, it is recommended to incubate for 7 days at 60 °C.

D-value: calculated at 50 °C, 3.500 ppm of peracetic acid.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %. Sterility conditions are necessary only during inoculation process performed in laminar flow.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep away from sunlight in its original box.

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10-30 °C.

Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

Shelf life

2 years.

Packing

100 envelopes per aluminum foil bag.

*ATCC® is a registered trademark of American Type Culture Collection.

Infection Control Division | Biological Indicators

BT400 Spore strip biological indicator

For liquid peracetic acid high level disinfection processes

Labelling

On product's packing: Product code and description, storage conditions, process for intended use, regulations and standards data, batch number, bacterial load and strain, manufacture and expiration date, presentation, manufacturer information, bar code and Data Matrix.

Possible target markets

Healthcare and Industry.

Other important information

It is advisable to incubate at 60 °C in Bionova® IC10/20 incubator. Read product's instruction for use thoroughly before use.

Precautions

Do not store the product near disinfecting agents.
Do not use this product to control radiation, ethylene oxide or any disinfection processes other than liquid peracetic acid.

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