Infection Control Division | **Cleaning Indicators**

Lumenia CDWE

For endoscope cleaning performance tests in Automatic Reprocessing Washing Machines

Usage

Designed to be used with Chemdye® Lumenia CCDER devices to evaluate endoscope cleaning performance in automatic reprocessing washing machines.

Applicable regulation

Designed under Quality Management System standard ISO 13485:2016/NS-EN ISO 13485:2016.
ISO/TS 15883-5:2005.

Characteristics

Synthetic, self-adhesive and temperature-stable support printed with indicator colored ink. Allows detection of all factors affecting the outcome of the cleaning process of endoscopes in Automatic Reprocessing Washing Machines.

Strip size: 38 x 6 mm (approximate); 20 strips per sheet.

Sheet size: 98.5 mm x 80 mm.

Initial Color: fuchsia

Result: elimination of fuchsia organic substance.

To interpret results visually, use results reference guide on products' instructions for use.

Optionally, the readout of the cleaning indicator can be done using Trazanto Automatic Quality Control and Traceability System (see instructions for use in device's manual).

NOTE: If unfavorable results are obtained, check quality/quantity of chemicals, water pressure, temperature, cleaning time and general condition of the washing machine.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, store in dark place within its original package.

Transport conditions

Storage conditions should be strictly followed.

Transport in closed and strengthened boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Shelf life

5 years.

Infection Control Division | **Cleaning Indicators**

Lumenia CDWE

For endoscope cleaning performance tests in Automatic Reprocessing Washing Machines

Packing

L1 and L2: 120 units of cleaning indicator per box (6 sheets per bag).

L122: 360 units of cleaning indicator per box (18 sheets per bag).

LSF1: 720 units of cleaning indicator per box (12 sheets per bag, 3 bags).

Labelling

On product's label: product's code and description, applicable regulations, batch number, expiration date, product's presentation, Trazanto logo and datamatrix code.

On product's bag: product code, storage indications.

Possible target markets

Healthcare and Medical Industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Store the product within its' original package, away from light, disinfectants, gases and humidity.

Do not re-use the product.

Do not touch indicator ink with your hands.

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