Infection Control Division | Chemical Indicators

CD16 Multivariable Process Indicator

For monitoring Ethylene Oxide sterilization processes

Usage

United States

Terragene Chemdyne® CD16 is a chemical process indicator intended for monitoring the efficacy of ethylene oxide sterilization processes. The chemical indicator changes from purple/brown to green to indicate that the conditions of the cycle have been met. For Ethylene Oxide sterilization cycles: 600 mg/L, RH 60 %, 55° C, 240 minutes.

Outside the United States

Monitoring the efficacy of ethylene oxide sterilization processes. Pass conditions: 600 mg/l EO, RH 60 %, 30 °C, 30 minutes // 600 mg/l EO, RH 60 %, 54 °C, 20 minutes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016. ISO 11140-1:2014.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

FDA 510(k)

K191021.

Characteristics

Type 4 Chemical Indicator (according to ISO 11140-1).
Double punched card strips printed with indicator ink.
Size: 200 x 15 mm.
Initial color: **Purple/brown.**
Final color: **Green.**

100 % Toxic Heavy Metal free

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep in a dark place in its original box.

Infection Control Division | Chemical Indicators

CD16 Multivariable Process Indicator

For monitoring Ethylene Oxide sterilization processes

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10 - 30 °C.

Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

Shelf life

5 years.

Packing

250 strips per aluminum foil bag.

Packing information: product description, instructions for use, storage conditions and manufacturer information.

Labelling

On product's packing: product code and description, process for intended use, classification according to regulation, color change, product presentation, batch number, expiration date, barcode and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to Steam, Dry Heat or any sterilization processes other than EO.

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