Infection Control Division | **Process Challenge Devices**

PCD26-C Process Challenge Device

Test pack for Steam sterilization processes

Usage

United States

Terragene® Integron® PCD26-C Process Challenge Device provides a defined challenge resistance against the claimed cycles shown below and demonstrated resistance equivalence to the ANSI/AAMI 16 towel pack. The device provides routine monitoring and sterilizer qualification testing steam sterilization processes.

Autoclave/Steam Cycles
Gravity Displacement
121 °C
30 min
Dynamic Air Removal (Vacuum Assist)
132 °C
4 min

Outside United States

Terragene® Integron® PCD26-C Process Challenge Device has been designed for a rapid and easy monitoring of dynamic air removal steam sterilization processes at 132/135 °C ≥ 4 minutes and gravity air displacement processes at 121 °C ≥ 30 minutes.

Applicable regulation

Designed under Quality Management Systems standards ISO 13485:2016/NS-EN, ISO 13485:2016. ISO 11140-1:2014. ANSI/AAMI ST79.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

FDA 510(k)

K191021.

Characteristics

Porous card system holding an IT26-C Chemical Integrator (Type 5 according to ISO 11140-1:2014). The system also contains a self-adhesive Record Card, PCD26-C-RC, where data on sterilization cycle may be registered.

Cardboard box (10.5 cm x 6.5 cm x 1.8 cm approximately) with a Process Indicator printed on it (Type 1 according to ISO 11140-1:2014).

Process Indicator:

Initial color: **blue**.

Final color: **dark grey or black**.

Infection Control Division | **Process Challenge Devices**

PCD26-C Process Challenge Device

Test pack for Steam sterilization processes

Integron® IT26-C Chemical Integrator (within the challenge device):
The migration is visible through a zone marked ACCEPT or REJECT, thus indicating whether sterilization conditions were met.

Integrating condition is calibrated against the kill time of 10^6 *Geobacillus stearothermophilus* ATCC 7953** spores' population, calculated in BIER (Biological Indicator Evaluator Resistometer).
Conditions: saturated steam at 121 °C, 128 °C and 135 °C.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10-30 °C.

Products should be transported in closed and reinforced boxes in order to avoid damages.

The transport of this product does not represent any risk for human health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

Shelf life

5 years.

Packing

Packing information: product code and description, regulation, process for intended use, conditions and instructions for use, storage conditions and manufacturer information.

Labeling

On the packing: product code, batch number of the PCD, batch number of Chemical Integrator, manufacture date, expiration date, barcode and data matrix code.

On PCD26-C-RC: product code, classification of the chemical integrator according to regulation and process for intended use.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

Other relevant information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to ethylene oxide (EO), dry heat or any sterilization processes other than steam.

*ATCC® is a registered trademark of American Type Culture Collection.

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